



# Eight Tips to Extend Your Alaris Medsystem III Pump's Life



888-509-8760

[www.aiv-inc.com](http://www.aiv-inc.com)

[aivsales@aiv-inc.com](mailto:aivsales@aiv-inc.com)

Infusion pumps are an essential part of hospitals, physician offices, outpatient surgery centers, emergency service vehicles and veterinary clinics. But what happens when they break down or just stop performing in peak condition after years of wear and tear?

Despite new models coming out each year from various manufacturers, your Alaris Medsystem III infusion pumps were built to provide years of safe and efficient operation. Here are AIV's top tips for extending the life of your Alaris Medsystem III infusion pump:

### **Tip 1: Replace the Batteries Regularly**

Sometimes, pump failure could simply be due to old batteries in your unit. The rechargeable batteries found in your Alaris Medsystem III hold a charge less effectively over time. Regular replacement of these batteries will lessen the chances of false error codes and inadvertent pump shutdown.

Second-source medical battery suppliers can also be just as effective as the old OEM battery, provided that the replacement battery is rated and constructed for use in specific IV pump models.

### **Tip 2: Replace the Infusion Pump Casing and Exterior**

Replacing the exterior parts of the Alaris Medsystem III not only protects against fluid invasion and other issues, it also visually presents the pump positively to the patients it is used on. These cases are available from AIV; [brand new lower housing \(AIV Part #: AL11026\)](#) for the Alaris Medsystem III on their website.

### **Tip 3: Repair/Restore Broken Pumps**

A malfunctioning pump does not necessarily mean it must be discarded. Many failures happen to parts that can be easily accessed and replaced, significantly lowering the department's overhead costs. There are many repair options available:

#### **a. Diagnose and Repair In-House**

Before anything else, try consulting your in-house professionals. You not only save time but money as well when you reach out to your resident technician versus outsourcing advice from a third-party service.

## **b. Outsource Repairs and/or Services to A Third Party**

Alternatively, you can also have your Alaris Medsystem III pump serviced by a third-party vendor like AIV that you trust and maintain a strong relationship with. AIV's technicians and staff can be very helpful with emergency repairs and other services. AIV provides a 120-day warranty on all new replacement parts, a 60-day warranty on recertified Alaris Medsystem III parts, and pump repairs with a 6-month warranty.

All AIV repairs are performed in their ISO 13485:2016 certified facility to strict quality standards. They offer both flat rate, all-inclusive pump repairs as well as fixed-rate labor repairs to suit your budgeting needs.

### **Tip 4: Attend to Warnings and Nuisance Alarms Promptly**

Attending to pump warnings and alarms immediately will keep your infusion device well-calibrated and in good working condition. The most common parts that may require recalibration or replacement often include the cases, LCD and AC adapters.

If you find that your Alaris Medsystem III needs to have its parts replaced, you can look into either purchasing used parts or new parts. Used parts are a more cost-effective option between the two. Keep in mind that for older pumps, this may be the only option. Make sure that any used parts you buy are tested and certified by the reseller to ensure proper function.

Most internal components of an infusion pump can be replaced, giving you a budget-friendlier alternative to purchasing brand new equipment. AIV supplies both [new parts for the Alaris Medsystem III](#), which include a warranty, 120-days for new replacement parts.

### **Tip 5: Perform Regular Service (PMs)**

Regular pump service can proactively identify and mediate issues that may cause the pump to malfunction. Alaris Preventive Maintenance covers inspection for physical damage, battery level, and adapters. It also includes calibrations procedures.

## Tip 6: Maintain and Clean Power Cords & Adapters

Not only can faulty wiring cause damage to the pump itself, but it can also make it unsafe for clinical staff and patients to use. Ensure all electrical cords and adapters are clean and damage-free through regular cleaning and maintenance. Early detection of cable issues may enable the repair instead of replacement of the part. Once faults are discovered however, be sure to replace them immediately. New power cord and [adapter units \(AIV Part #: AC10632\)](#) can easily be purchased from providers like AIV.

## Tip 7: Undertake Additional Infusion Pump Training

Regular education on pump function and common problems will enable your facility to diagnose and correct common pump problems. Many biomedical conferences, such as the [CMIA](#) or [AAMI](#) annual conferences, offer these seminars free of charge to biomedical professionals.

## Tip 8: Purchasing Additional Models of Equipment Required

Purchasing several models of the same pump can be a good insurance policy for your institution. Quantity discounts are available from some distributors, and the extra pumps can be used as substitute floor units, or for replacement parts.

Replacing your current machines with all-new equipment doesn't have to be the only solution! By following these top eight tips, you can add several more years of safe usage to your Alaris Medsystem III infusion pump devices and earn of additional savings for your facility.

For reliable, quality-driven Alaris Medsystem III infusion pump replacement parts and services, call on AIV. AIV manufactures and recertifies a range of power adapters, LCDs, cases, and other high quality parts have been designed to strict quality standards in our US-based ISO 13485:2016 certified facility. In addition, all Alaris pump maintenance and repairs are performed by AIV's trained and experienced technicians. This is why AIV's Alaris Medsystem III pump replacement parts and Alaris infusion pump repairs are trusted by hospitals worldwide.



Need more help with your Alaris Medsystem III infusion pump?  
Contact AIV today at **888-509-8760** or send an email to [aivsales@aiv-inc.com](mailto:aivsales@aiv-inc.com).

**888-509-8760**  
[www.aiv-inc.com](http://www.aiv-inc.com)  
[aivsales@aiv-inc.com](mailto:aivsales@aiv-inc.com)

**7485 Shipley Avenue**  
**Harmans, MD 21077**